



InspiroBio COVID-19 Antigen Rapid Test Results in 15 minutes using an anterior nasal swab.

Test Details

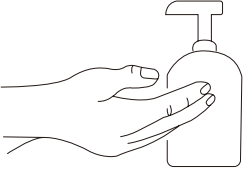
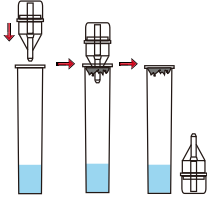
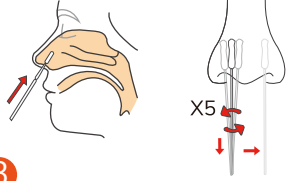
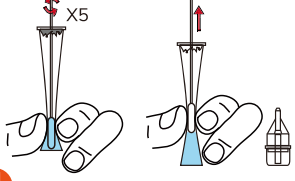
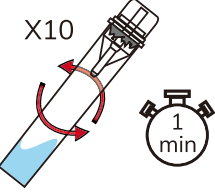
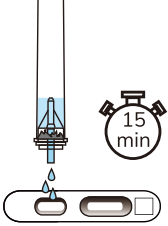
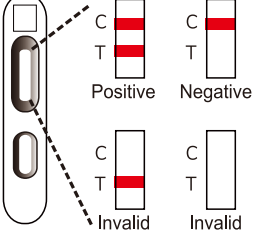
The InspiroBio COVID-19 Antigen Self-Test is an easy-to-use and reliable test device highly sensitive to low viral load variants such as Omicron. The unique spill proof tube with closing design allows users to comfortably self-collect samples using a sterile swab, get the result in just 15 minutes. The tests come in package sizes to perform 1, 2, 4, 10, 20 test(s) per box to meet the needs of various user groups and serial testing.

Features and Benefits:

- Result in 15 minutes.
- Unique spill proof tube with closing design.
- Validated through strict EUA clinical study amid Omicron dominance.
- High sensitivity and specificity on asymptomatic patients.
- Ability to detect extremely low viral loads.
- Built in tube and cap holders in the kit box.
- Capacity of high volume production.
- 18-months shelf life.



Test Procedure

<p>Please clean your hands and table. Read the product insert and quick user guide carefully.</p>  <p>1</p>	<p>Take out the test cassette, pierce the sealing membrane of lysis buffer, set them aside for later use.</p>  <p>2</p>	<p>Take a specimen collection swab and insert it into the nostril about 2-3 cm for scraping 5 times. Repeat this process at the other nostril to ensure an adequate sample size is collected.</p>  <p>3</p>	<p>Insert the specimen into the lysis buffer, squeeze the tube and stir the swab 5 times. Keep squeezing until all liquid removed from the swab. Return the swab to its bag for later disposal.</p>  <p>4</p>
<p>Put on the nozzle cap and shake the tube gently 10 times. Let stand for 1 min.</p>  <p>5</p>	<p>Add 3 drops of specimen vertically into the sample well, let stand for 15 min.</p>  <p>6</p>	<p>Result interpretation.</p>  <p>7</p>	<p>Watch WESAIL operation video online.</p>

Caution:

1. Please read the product insert and quick user guide carefully.
2. Please avoid directly touch or contaminate the tip of the nozzle cap.

To InspiroBio Scientific Inc.

Address:: 1233 BOCKMAN RD., #17, SAN LORENZO,
California 94850, USA

Email: market@inspirobio.com

InspiroBio